



# TUB MOUNTED SLIDING TUB TRANSFER BENCH


**Do Not** install or use this equipment without first reading and understanding this instruction sheet.

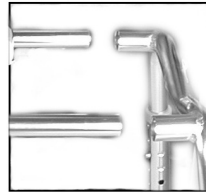


Fig, 1

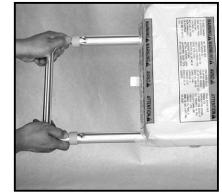


Fig, 3

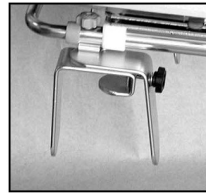

Fig, 4

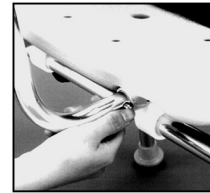

Fig, 5

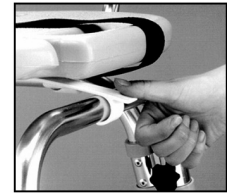

Fig, 6

1. Install the 2 adjustable leg extensions onto the 2 legs of the frame tubes.
2. Snap the side frame onto the sliding rails (Fig. 1)
3. Slide the seat all the way to the side handle bar. Insert the U-shaped tube into the other end of the frame and line up the holes on the frame and the U-shaped tube. (Fig. 3) Unscrew the nut knobs on the tub clamps, remove the screws, and reverse them. Insert the screws on the tub clamps through the holes on the frame and U-shaped tube (from bottom to top). The tub clamps must be positioned so that the black locking screw will be on the inside of the tub and the off set space on top of the clamp is outside of the sliding tubes on the frame. (Fig. 4) Attach the nut knobs but do not tighten yet.
4. Peel the paper off the rubber pads and stick them on the inside of the clamps at the points where contact with the tub will be made. Install the sliding bench onto the tub. Adjust the clamps and the leg height on the other end of the bench.
5. Turn the suction cups to the proper position to make full contact with the bottom of the tub and tighten the black locking knob on the clamps. Then tighten the nut knobs on top of the clamps.
6. Insert the backrest assembly onto the back of the seat. The two back braces fit into the tubes beneath the seat. Press the two brass locking pins in and push the back braces in until they click into place. (Fig. 5)

## **USE**

1. Slide the bench seat all the way to the outside edge of the tub and lock it so that it will not slide (press down the white plastic locking mechanism until it snaps onto the U-Shaped Tube). (Fig. 6)
2. Press down the red handle under the seat to turn the seat 90 degrees to face the patient. After pressing the red handle turn the seat slightly, then release the red handle and turn the seat until it locks. (Holding the red handle down will not allow the seat to lock and may cause instability). Once the seat is locked in position assist the patient onto the seat and attach the safety belt. To turn the patient into the tub release the seat by pressing the red handle down again as you turn slightly. Then release the red handle and continue turning the seat to position the seat and patient facing forward. Release the white plastic lock and slide the seat to the inside of the tub and lock again on the other side.