

#41011 **2-STAGE TILT TOP OVERBED TABLE**

FEATURES

1. Top height can be easily adjusted from 700mm to 1100mm (27½" to 43¼") by releasing knob.
2. One fixed top 200×384×20mm(8"×15"×3/4") and one tilt top 600×384×20mm(23½"×15"×3/4"). Both of them are particle board tops.
3. Four easy rolling φ40mm casters provide smooth and stable mobility.
4. Tilt function allows user to turn the table to preferred angle for reading.

INSTALLATION

1. Screw in four φ40mm casters on to two base legs.
2. Assemble two base legs with the outer mast frame.
3. Assemble the inner mast (with tilt mechanism) with the fixed top.
4. Insert the inner mast with tilt mechanism and fixed top into the outer mast frame. Then, tighten the height control knob.
5. Slide the tilt top into the tilt mechanism of inner mast, and tighten the tilt control knob.
6. Table top heights can be adjusted by releasing the height control knob. Tilt top can be turn to desired angle by releasing the tilt control knob.

WARNING

1. Do not lean on the suspended end of table top at any time. This may tip over the whole overbed table.
2. Do not put cigarette on the table top as it may catch fire.
3. Make sure the tilt control knob and height control knob are tighten at any time. Otherwise, the goods seated on tops may tip over.

MAINTENANCE

1. Periodically check fit and tightness of all nuts, bolts, and adjustment knobs to make certain table is safe for use. Gently wipe the overbed table with a damp, soft cloth to remove dust and dirt.
2. To wash off more stubborn grime, clean with mild soap and water. Do not use cleansers that may damage the finish. Be certain to thoroughly dry overbed table with a soft cloth.